

Rotem Industries is proud to announce the official release of MediSmarts version 5.0. Over 110 users worldwide have contributed their ideas into the latest version. We have spent over 1 year perfecting the software to match and better the stability, and ergonomic features of previous versions to match the long lasting, sturdy hardware provided since the 80's. Here are some of the new features of this version:

## New Features

Compatible to Win XP, 2000 and Version 7.  
 Network compatible over intranet and internet  
 Multilingual Support  
 More stable and user-friendly

## Highlights

The Life Sign  
 Goodbye to Archives  
 Improvements to the Graph  
 Enhanced Local Setup  
 New improved Installer password  
 European Alarm Point system introduced


## The Life Sign



This beating heart appears in the left hand corner of your screen and signifies that your computer is alive and kicking!! If the heart stops beating your computer has frozen and the green points on your map are displaying old, irrelevant data. Users can now glance at the MediSmarts screen to verify that all points are green and that the computer is alive.


## Goodbye to Archives

From this version onwards, each historical graph will include arrows to view the next or previous graphs. No need to open or close archives. In order to enable this feature to work properly and display the graphs in chronological order the names of the archive directories should be changed to two digit format: 2010-3-18-15 should be modified to 2010-03-18-15

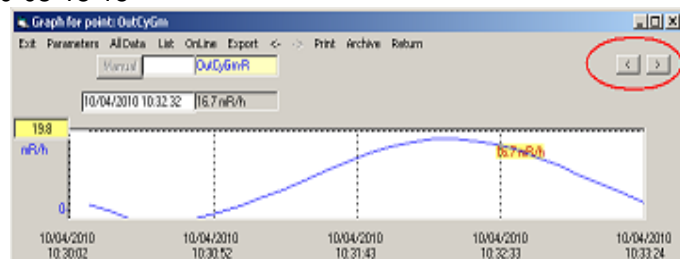

## Extensive work on the graphs is reflected in the following new features:

- Up to 5 graphs can be presented simultaneously
- Tooltips on mouse click showing values
- Each Graph displays its unique unit of measurement
- The maximum Y range (value) can be set per graph
- Right Click on any graph opens the REMOVE option and allows the user to remove this graph from the list
- The air flow data is not displayed as default but can be opened if required. Customers commented this was not useful.

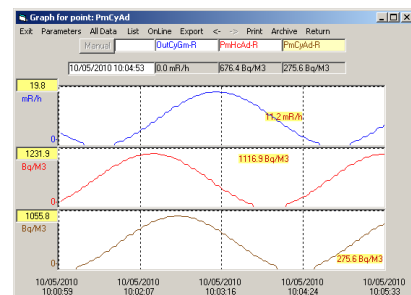

## Miscellaneous

- Enhanced local Setup Screen
  - European System is different to American system as follows:
    - In the Point Details screen - User decides which alert or alarm activates the solenoid in the DPU
    - User also decides which color activates an alarm. Only one alarm per point.
- New improved Installer password makes troubleshooting easier and quicker
- The Sol32.ini file is now situated in the MediSmarts Folder
  - Added capability of running more than one MediSmarts per computer
  - IMPORTANT - For upgrades or new installations, delete sol32.ini from the Windows folder.